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## ETOILE

### BLUE ENERGY

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*Combined instrument for lifting and skin rejuvenation*

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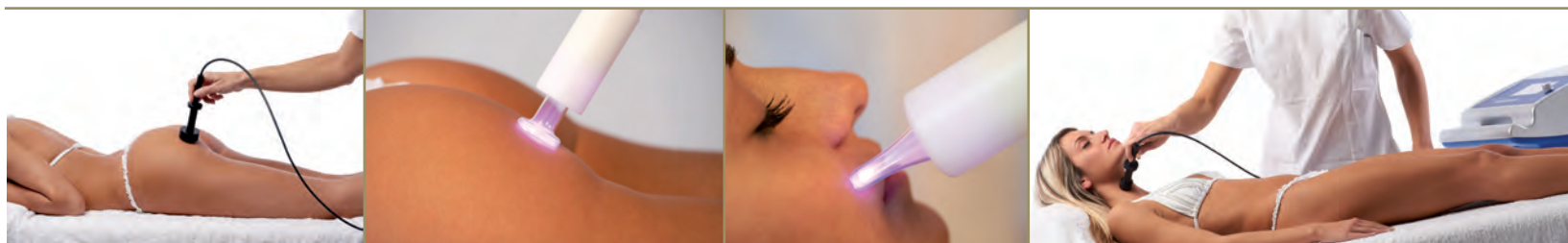
## TECHNICAL FEATURES

### General Technical features:

Secondary circuit electrical tension: 15V  
Digital totalizer timer  
Supply voltage: 230V 50-60Hz  
Dimensions: 65x36x19 cm  
Weight: 13 kg.  
Total absorbed power: 180 VA  
9 Selectionable levels  
Visualization by monochromatic graphic display 125x70 mm  
Control by microprocessor  
Equipment for combined treatment: 2 applicators with different size, each for both treatments

### Radiofrequency technical features:

Working frequency: 1,5 Mhz  
Overload protection  
Total electrical insulation between the device and the patient  
Maximum absorbed power by radio-frequency amplifier: 80 W  
Equipment: 3 applicators each for small and large areas, characterized by capacitive coupling with high impedance (then without need of return electrode)

### Vacuum technical features:

Continuous measurement and indication of vacuum level  
Equipment: set of cupping glasses for suction and set of cannulas for comedones

Blue Energy is a complete device which has been specifically studied in order to effect combined treatments by using two systems widely known: radiofrequency and endermology massage performed with a pulsed vacuum action. This device has been planned in order to fight against skin ageing, especially of the face, and as valid help for revascularization of certain areas of the body where, because of a a blood circulation stasis with association to advanced age and exposure to the sun, there has been a slight blemishes formation (wrinkles, scars, stretch marks...). The two systems can be used in sinergy or individually and, to this end, Blue Energy is provided with specific applicators in order to effect the treatments in both ways. Particularly the combined treatment, effected with only one handpiece, allows to halve the working times with big advantage for the operator. Radiofrequency allows to send energy by generating internal heat where it is specifically required with advantage on local blood circulation and a subsequent better derma cells irroration. Best advantages on blood circulation is obtainable by coupling the two systems in sinergy. Endermology massage, performed with a pulsed vacuum action, allows to effect a useful detaching action.

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